

Emis par le service:	Approuvé par:	Date:	Révision:	Page:
MKT	A. Schreyer	24.04.2017	2	1

Operating instructions:

The turbines, drills and hand pieces must be in perfect condition.

Introduce the instruments carefully and without forcing into the adjustable clamp. The miniaturised instruments

should be held in the clamp up to the end of their cylindrically shaped section.

The required operating pressure is situated between 0.3 and 2N (30-200p). Excessive pressure must be avoided at

all costs.

The recommended rotating speeds as noted on the packaging and in the accompanying table must be closely

adhered to.

During preparation, avoid at all costs any blockage or levering action. Immediately eliminate any instruments that are twisted or rotate in a non-concentric fashion.

Make sure that the working part of an instrument being used is kept sufficiently cooled (a minimum of 50ml/min), any deviation of the spray risks causing thermal lesions.

Hygienic recommendations:

Before being used for the first time and immediately after use, the instruments should be disinfected, cleaned, dried and sterilised.

Respect the time periods and methods of disinfection as well as the temperatures used in the sterilisation process.

Before sterilising, place the instruments in appropriate containers.

After sterilising, store the instruments in a dust free environment, either on brackets or in containers. Any surgical instruments should be kept in closed containers.